


Micro 10⁺



Disinfection, cleaning and care for all dental and surgical instruments.

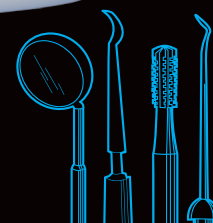

Economical, aldehyde-free concentrated solution for pre-sterilisation disinfection and cleaning of all dental and surgical instruments, including burs, root canal instruments, forceps, probes, mirrors, etc. Micro 10⁺ has a broad spectrum of efficacy and can be used in a soaking tank and ultrasonic cleaner.

Key product features

- Very economical 2% dilution
- Contact time : 15 minutes
- Suitable for all instruments
- Contains corrosion inhibitors to protect instruments against rusting
- CE marked – Complies with Medical Devices Directive 93/42/EEC
- VAH/DGHM certificated



USF Healthcare S.A. | Rue François Perréard 4 | CH-1225 Chêne Bourg | Geneva | Switzerland
Tel + 41 22 839 79 00 | Fax + 41 22 839 79 10 | info@usfhc.com
www.usfhealthcare.com


UNIDENT
SWISS

Micro 10⁺


Protocol for use

Before use

Ensure appropriate gloves are worn.

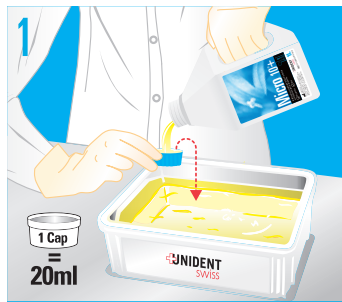

Dilute Micro 10⁺ in a soaking tank at 2% (20ml per 1 litre of water).

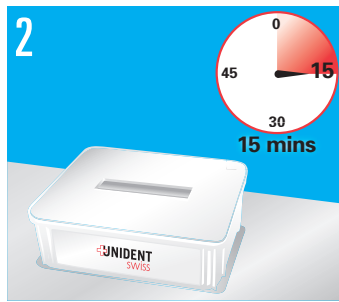


Immerse the instruments in the solution and leave for 15 minutes.



Carefully remove the instruments from the solution and inspect. Brush if necessary to remove any remaining visible debris.



Rinse instruments under running water or by immersion in clean water.

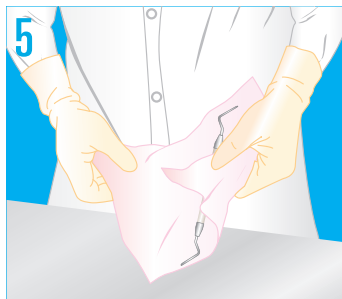

Dry instruments with a clean disposable paper towel. Pack the instruments and prepare for sterilisation.


Recommended dilutions

- 2% (20ml in 1L water) for all dental instrumentation
- 5% (50ml in 1L water) for ultrasonic cleaning: 3 to 8 minutes. Supports heating up to 50°C.
- 10% (100ml in 1L water) for small instruments, e.g. canal instruments, diamond burs, etc.

Precautions

Dangerous - respect the precautions for use (established according to the Directive 99/45/CE and its adaptations). Storage between +5°C and +35°C. Medical Device Class IIb, CE0459 (Directive 93/42/CEE modified).

Efficacy

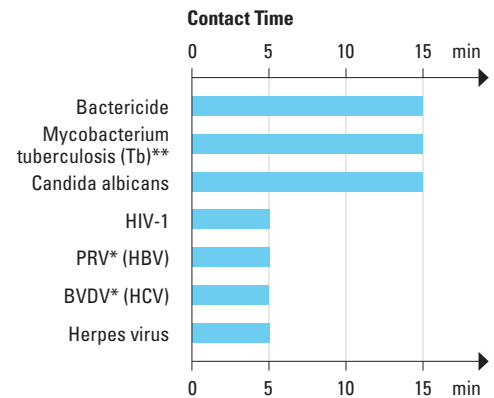

* PRV: Surrogate of Hepatitis B
* BVDV: Surrogate of Hepatitis C
** Dilution 5%

Conforms to European standards

Bactericide: EN 1040, EN 13727, EN 14561

Yeasticide: EN 1275, EN 13624, EN 14562

Product composition

100 g of Micro 10⁺ contains 9 g of alkylbenzyltrimethylammonium chloride, amphoteric and non-ionic surfactants, complexant, corrosion inhibitor and additives.

Physical data

Colour: Pale yellow

Perfume: Without perfume

pH: 7.5 (diluted at 2%)

Shelf Life: Concentrate 3 years

Diluted, unused 28 days

Diluted, used 7 days depending on soil quantity

Container: Made of recyclable HDPE material

Packaging

| 1L | 2,5L | 5L |
|----|------|----|
| ✓ | ✓ | ✓ |

Accessories

3L Soaking tank (INTANK)

Soaking tank for burs



ECOEngage
Our Ecological Engagement



USF Healthcare
Unident • Steridine • Farmec

UNIDENT
SWISS