

Tegaderm™ IV Transparent Film Dressings



Secure.
Protect.
Stabilize.

IV Site Care




Enhanced Security

- Notched design for a better seal and reduced edge-lift around the catheter
- Sterile tape strips for anchoring hubs, lumens or tubing
- Enhanced seal and secural helps to maintain a barrier from outside contaminants

Performance Benefits

- Specially designed for many central catheter applications including the difficult to dress jugular site
- Soft cloth notch provides added securement for bulky catheters including dialysis catheters
- Soft cloth notch may eliminate the need for a separate hub cushion
- Unique picture frame delivery
- Pre-cut sterile tape strips offer added convenience and versatility

3M™ Tegaderm™ IV Transparent Film Dressing 1633/1635 Application and Removal Techniques

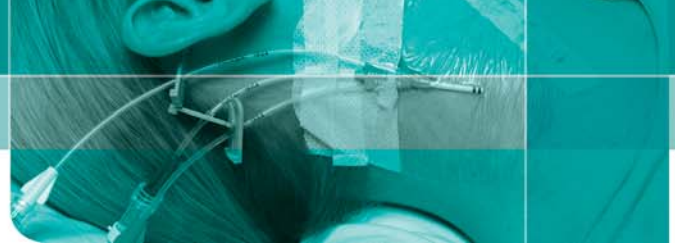

Application



1. Allow all preps and protectants to dry thoroughly before dressing application. Open package and remove sterile dressing. Remove the centre panel and place on sterile field. The tape strips will remain attached to this piece. Peel the liner from the dressing, exposing the adhesive surface.



2. Position the transparent portion of the dressing so that it is centred over the insertion site, while holding the notched portion off the skin.



3. Position the dressing so the notch fits snugly around the catheter port and under the hub. The notched portion of the dressing will help to secure the catheter. Slightly overlap the tabs under the catheter to form a tight seal around the catheter or lumens.



4. Press the dressing into place.



5. Slowly remove the frame while smoothing down the dressing edges. Next, seal the edges of the notch and smooth the dressing from the centre toward edges using firm pressure to enhance adhesion.



6. Sterile tape strips can be used under the catheter wings or hub to protect the skin, over the catheter wings or hub to enhance catheter stability, or to secure IV tubing or stabilize catheter lumens. Fold edge of sterile tape strip over itself making a small tab for easier removal. Record information on label, remove from frame then place on or near the dressing.

Removal



1. Remove any tape strips applied on top of dressing. Saturate with alcohol, if necessary, to assist in loosening the tape strips.



2. Separate dressing tabs and gently peel back, or stretch to release the transparent portion.



3. Avoid skin trauma by peeling the dressing back, rather than pulling it up from the skin. Or, gently pull the transparent portion straight out to stretch it and release adhesion. A medical adhesive remover can also be used.

Ordering Information

3M™ Tegaderm™ IV Transparent Film Dressing 1633/1635 with label and securing strips



Product Number	Size	Dressings/Box	Boxes/Case
1633	7,0 cm x 8,5 cm 2¾ in x 3 ¼ in	100	4
1635	8,5 cm x 10,5 cm 3½ in x 4 ¼ in	50	4



**Health Care
3M Canada**
Post Office Box 5757
London, Ontario N6A 4T1
Canada
1-800-364-3577
www.3M.com/ca/healthcare

For more information, contact your 3M Skin Health representative or call the 3M Health Care Customer Helpline at 1 800 364-3577.

**Medical Division
3M Health Care**
3M Center, Building 275-4W-02
St. Paul, MN 55144-1000
USA
1-800-228-3957
www.3M.com/healthcare

3M™ Tegaderm™ SIMPLE. DEPENDABLE. TRUSTED.
Dressings

3M™ Tegaderm™ IV Transparent Film Dressings provide health care professionals with a single source for every IV catheter secure need. For more information, please contact your 3M Health Care representative or 1-800 364-3577.

3M and Tegaderm are trademarks of 3M. Used under license in Canada. Please recycle. Printed in Canada. © 3M 2008. All rights reserved. 0807-2732-E