

# SOF® TOURNIQUET GEN 4

Part#: SOFTT-W | NSN#: 6515-01-587-9943

US PATENT-8,303,620 and D649,642 and 7,776,064

## INSTRUCTIONS FOR USE


### To Apply the Tourniquet

1




**Slide Over Limb**

2

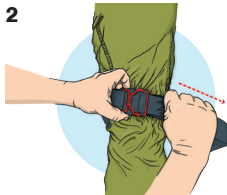

**Pull Tail Tight**

3

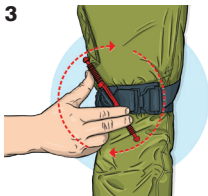

**Twist Handle**

4



**Secure in Clip**

5




**Lock in Triangle**

6



**Note Time**

**Applying the Tourniquet if Unable to Slide Over Limb**  
Follow steps 1b-4b

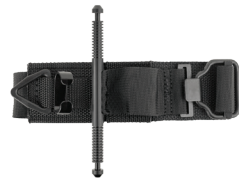

1b




**Lift Buckle**

2b




**Twist to Disconnect**

3b

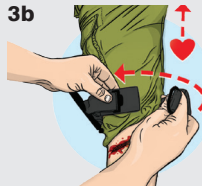

**Go Around Limb**

4b

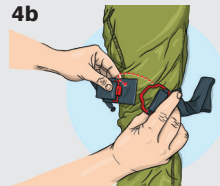

**Connect Buckle, Then Go To Step 2**

### SOF® Tourniquet GEN 4

For control of severe bleeding.

### SOF®T Storage

Store the tourniquet with the buckle connected and enough slack to slip the tourniquet loop over your limb.

TACTICAL MEDICAL SOLUTIONS, LLC

PHONE 864.224.0081

TOLL FREE 1-888.822.6331

FAX 864.224.0064

E-MAIL [info@tacmedsolutions.com](mailto:info@tacmedsolutions.com)

WEB [tacmedsolutions.com](http://tacmedsolutions.com)

1250 Harris Bridge Road, Anderson, SC 29621

