


Follow these Instructions step-by-step:

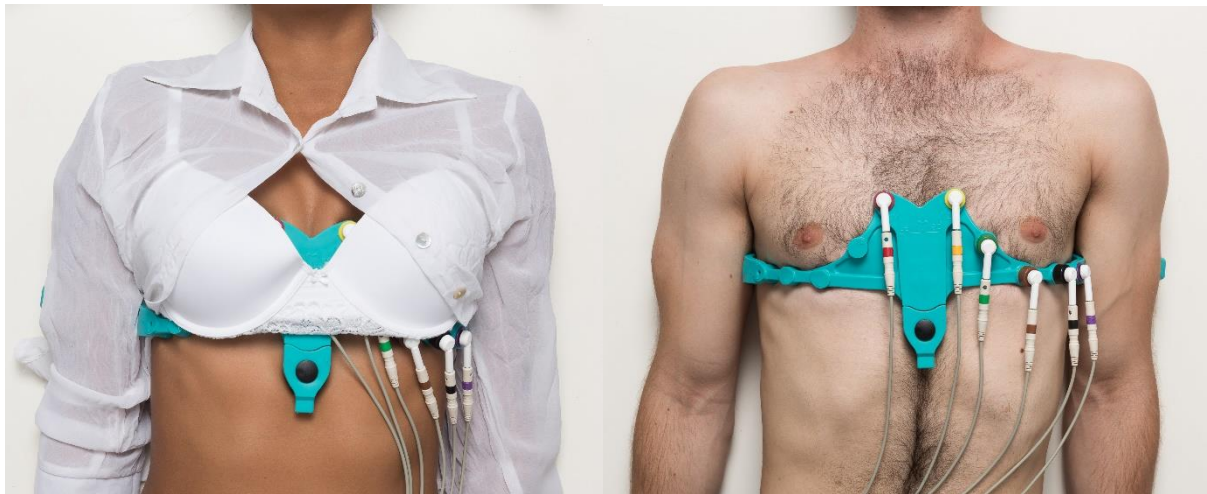

LevMed AllBrand™ ECG Belt with Handles

**Ensure the ECG Patient Cable is
connected to the ECG Device.**

Ensure the screws on the ECG Belt are tightly closed:



Connect the ECG Patient Cable to the screws using Snap/Crocodile connectors:

Banana to snap adaptor



Banana to crocodile adaptor



*If your ECG Patient Cable has Banana connectors, use the banana-to-snap or the banana-to-crocodile adaptors.


Connect the ECG Patient Cable to each of the 4 clamps:

For Banana Patient Cable:




1. Open the screw.
2. Insert the banana connector
3. Close the screw.

For Snap/Crocodile Patient Cable:


1. Close the screw.
2. Connect the snap/crocodile.

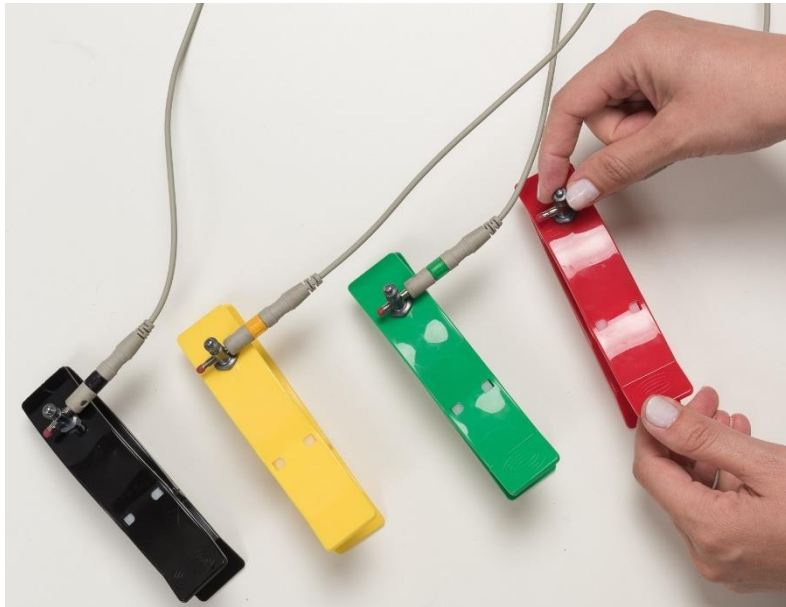

Open or Remove the patient's shirt:

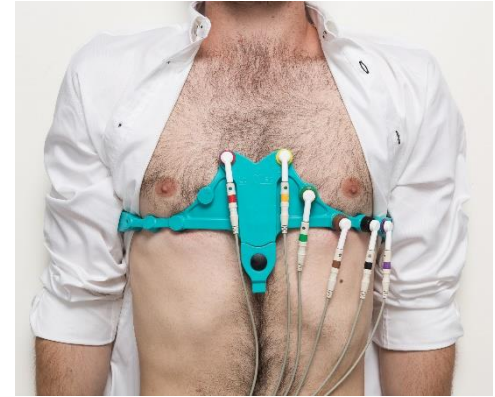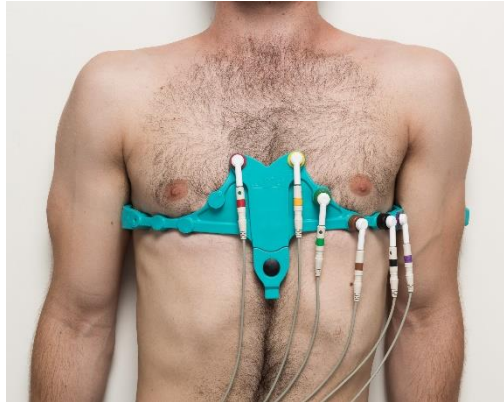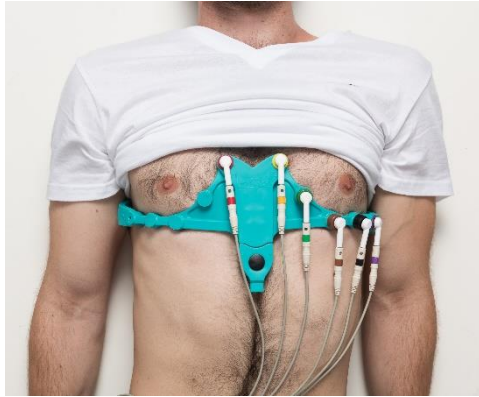

Wet the 6 Electrodes (V1-V6) with Tap water:

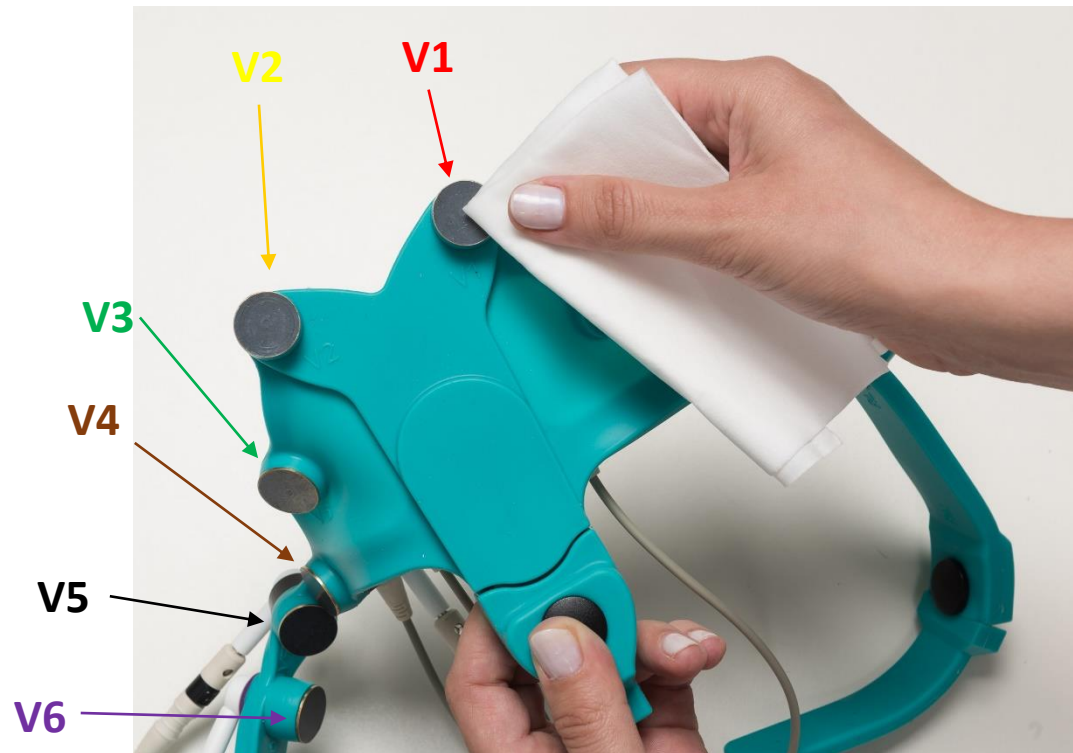


Adjust the right & left handles to fit patient's arms:



If necessary, bend V1 & V2 to ensure electrode contact with patient chest:



When the patient is lying down comfortably, apply the belt on patient chest with V1 and V2 symmetrically on both sides of the sternum:

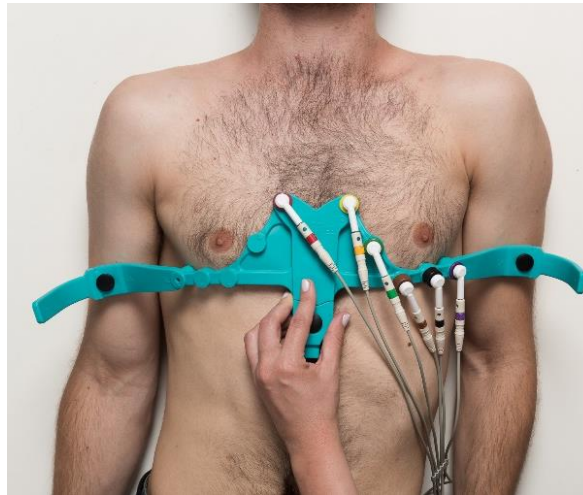

Note: for women, the belt may be applied under the bra:



**Apply the handles on mid-arms,
between elbow and shoulder:**




**Left
Handle**





**Right
Handle**

**Confirm electrodes V4, V5, and V6
are located beneath the breast:**

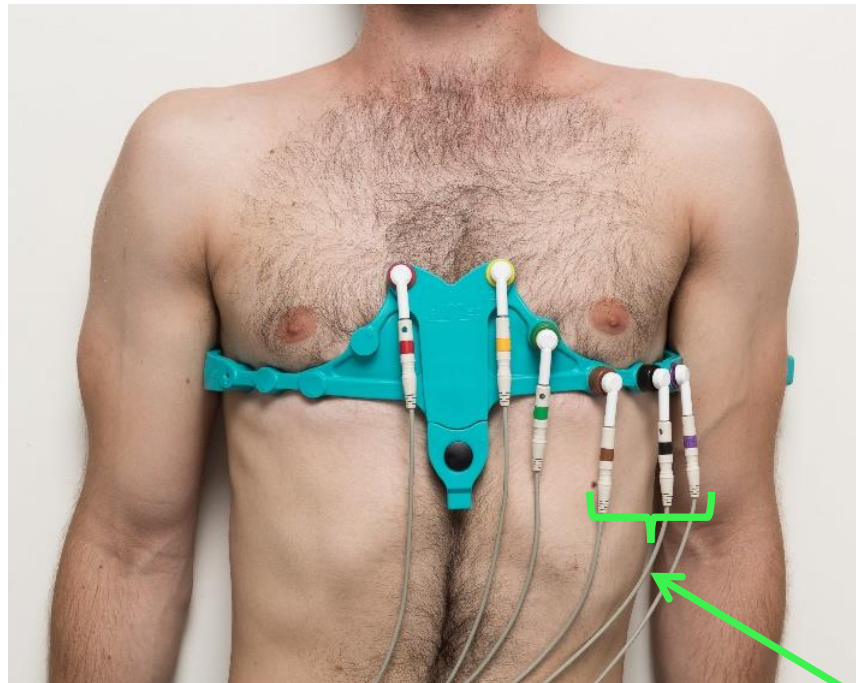


V4, V5, V6

Ensure all 6 electrodes are in contact with patient skin:

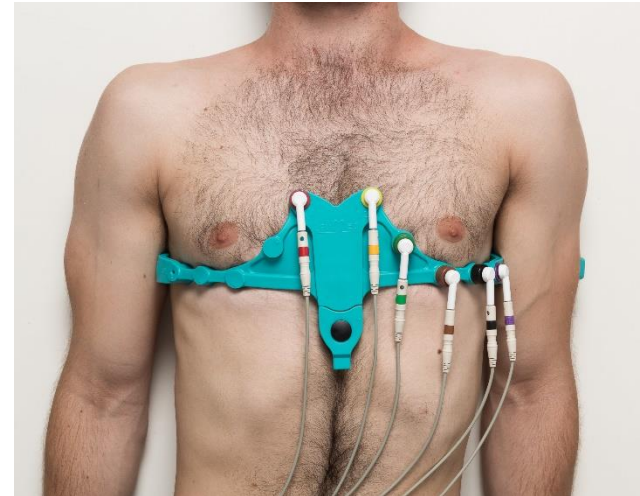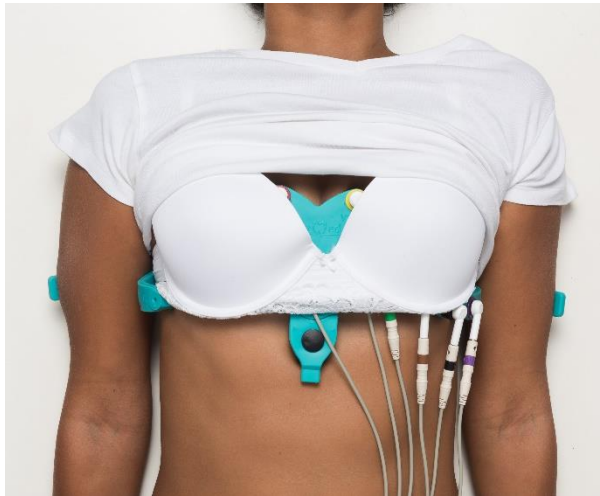

**Wet the Electrode on each of the 4 Clamps
with Tap water:**

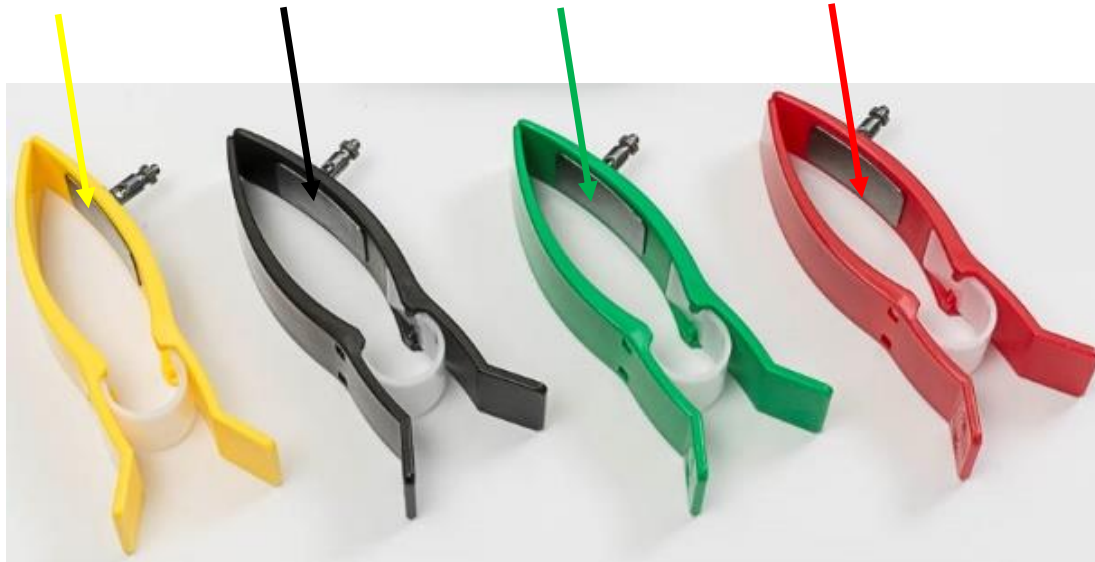

Apply the clamps on the limbs:

Red = Right Arm

Yellow = Left Arm

Green = Left Leg

Black = Right Leg




Ensure the patient is relaxed and does not strain his muscles.



Ask the patient not to move or talk during the assessment.



Note: in case more comfortable, the assessment may be done while the patient is in a sitting position:

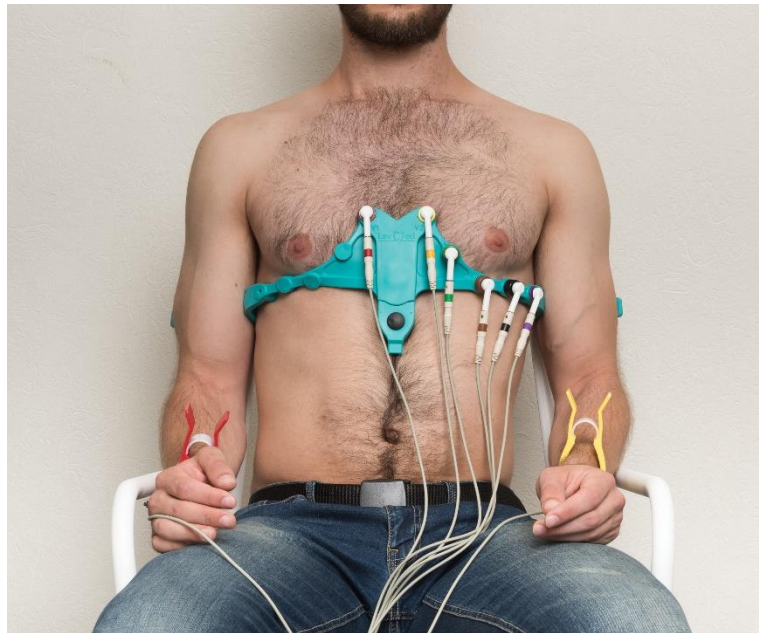

Press **'Start'** on the ECG Device to take ECG Reading.

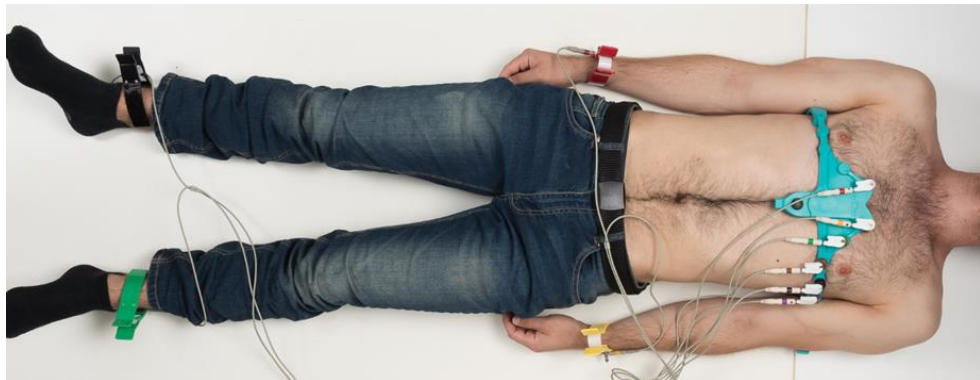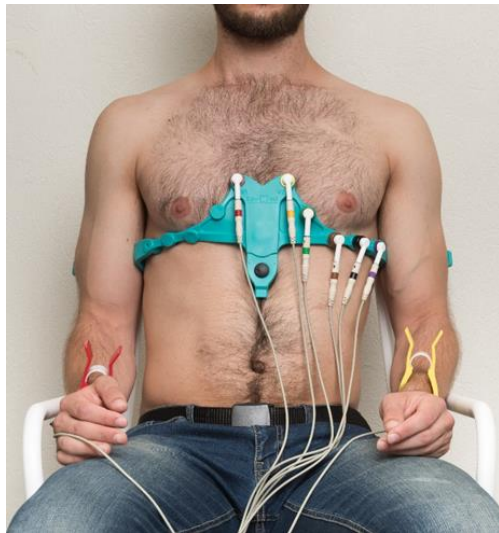

Troubleshooting

In case noise is seen on the ECG signal, ensure:

1. Water is applied on all electrodes.
2. All electrodes are in direct contact with patient skin.
3. Patient is relaxed, muscles not strained.





**Thank you for using LevMed
AllBrand™ ECG Belt!**

For more information contact us at info@levmed.net
www.levmed.net