


3M™ Tegaderm™ Paediatric Peripheral I.V. Dressing (1610)

- designed specifically with children in mind

Application Instructions



1 Peel open the package and remove the sterile dressing. Remove pre-cut sterile tape strips from carrier.

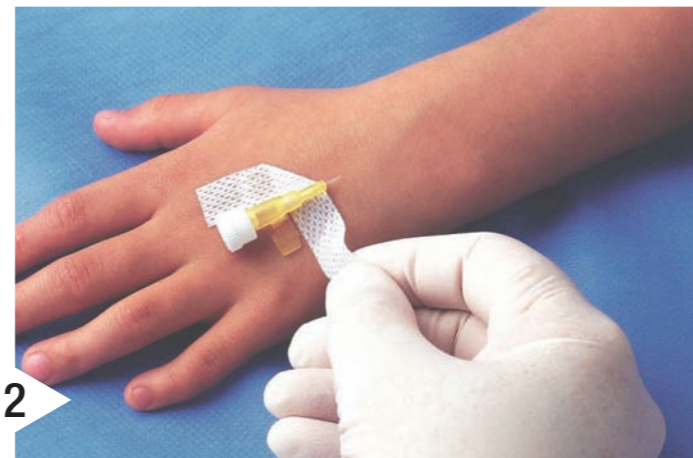

2 Apply sterile tape strips to secure cannula.



3 Place dressing over cannula so that transparent film is over insertion site. Position the soft cloth tape section over the cannula wings. Remove paper frame and smooth dressing edges to increase adhesion.



4 Tegaderm 1610 paediatric I.V. dressing may also be used with ported I.V. cannulae.



Removal Instructions

- Alternately stretch and release the film section parallel to the skin to break the adhesive bond
- Carefully peel back soft cloth tape section in the direction of hair growth
- Carefully peel off the tape strips